

Teeth Whitening Start Kit

Art no: 1200xx

Beconfident - Whiter teeth in 30 minutes

The Start kit contains the following items:

- A 10 ml tube with our Beconfident Sodium Bicarbonate gel
- 3 trays (one extra)
- Instruction sheet and a shade guide.
- App Download

How it works

There are several active ingredients that all work together. One of them is Sodium Bicarbonate. For the gel to work effectively, it must first penetrate the teeth. This is crucial for a good result and Beconfident's unique gel is quickly absorbed by the enamel and dentin. Our thermoform mouth trays are individually shaped through heating and can be reused minimum 5-6 complete treatments. Bleaching gel appears here with an enzyme found naturally in the saliva that releases oxygen, which bleaches both enamel and dentin. After a Start Kit has been used, a refill gel can be used to repeat procedure when necessary.



We recommend anyone to complete the entire whitening process until there is no gel left. Best is to do the cure 15 minutes for 10 or so consecutive days.

3 Unique Competitive Advantages

- Good Gel formulation with proven clinical results, supported by Beconfident intelligent App for best instructions.
- One extra mouth tray, which allows the user to shape two perfect individually, fitted mouth trays for best efficiency of the gel and to whiten all the teeth.
- Works safely for all users.

Ingredients

Glycerin, Sodium Bicarbonate, Aroma, Xylitol, Chondrus Crispus Powder, Polysorbate 20, Sorbitol, Aloe Barbadosensis Leaf Juice, Punica Granatum Extract, Chamomilla Recutita Extract, Sodium Flouride, Linalool, Limonene.

Frequent Asked Questions and answers

Is teeth whitening safe?

Yes, as long as it doesn't contain any peroxides or abrasives. Beconfident teeth whitening products are 100% peroxide free and instead uses a 32% Sodium Bicarbonate gel which does not have any side effects what so ever and is safe to use over and over again.

Group beconfident Europe AB

Beconfident- Top Quality Teeth whitening and Skincare systems designed in Sweden & Made in USA and Sweden
Makadamgatan 16, SE-254 64 HELSINGBORG, Sweden
Mobile: +46-(0)702- 56 50 00 and Office +46-42-22 22 92
SKYPE: johan.ling.vannerus,, E-mail: johan@beconfident.com
www.beconfident.info, www.beconfident.com, www.beconfident.se
Organization number: 556853-0041, VAT number: SE556853-004101, EORI-number: SE5568530041

beconfident[®]

Product Data Sheet



Product Data:

<i>Art No</i>	<i>Dimension WxDxH</i>	<i>Gross Weight (packaging included)</i>	<i>Net Weight (Product only)</i>	<i>Units Display Box</i>	<i>Units mastercarton</i>	<i>Unit EAN</i>
<i>1200xx</i>	<i>23 x 120 x 175</i>	<i>61g</i>	<i>37g</i>	<i>6</i>	<i>96</i>	<i>Local</i>
<i>CPNP Code</i>	<i>GMP ISO 22716</i>	<i>Country of Origin</i>	<i>Quality Test Pharmacies</i>	<i>Box bio- degradable</i>	<i>Pallet</i>	<i>RRP Euro incl VAT(+/-)</i>
<i>1536510</i>	<i>Follows</i>	<i>Sweden & USA</i>	<i>Yes</i>	<i>Yes</i>	<i>EUR Pallet</i>	<i>44,90 Euro</i>

Group beconfident Europe AB

Beconfident- Top Quality Teeth whitening and Skincare systems designed in Sweden & Made in USA and Sweden
Makadamgatan 16, SE-254 64 HELSINGBORG, Sweden

Mobile: +46-(0)702- 56 50 00 and Office +46-42-22 22 92

SKYPE: johan.ling.vannerus,, E-mail: johan@beconfident.com

www.beconfident.info, www.beconfident.com, www.beconfident.se

Organization number: 556853-0041, VAT number: SE556853-004101, EORI-number: SE5568530041